

SAMPLE COLLECTION & SHIPPING INSTRUCTIONS

Test Type	Sample Type	Tube Type
First Trimester		
Maternal Fetal Screen SM T1	Serum	Red Top or SST
Preeclampsia Screen SM T1	Serum	Red Top or SST
Second Trimester		
Sequential Screen FB <small>*Patient must have first trimester screen done through NTD Labs</small>	Serum	Red Top or SST
AFP Test (for ONTDs)	Serum	Red Top or SST
Quad Screen FB (stand-alone, or part of Sequential Screen FB)	Serum	Red Top or SST

Collection Instructions:

- For each test, collect 5 mL unspun whole blood (Red Top or SST) and ship the original collection tube with or without centrifugation.
- Place collection tube inside the outer screw-top tube to avoid potential for breakage or leakage of specimen in transit.
- Can be shipped at ambient or refrigerated temperature Monday through Friday.
- Specimens must be received at NTD Labs within 6 days of collection.
- Do not freeze.

CAUSES FOR REJECTION: Whole blood frozen, hemolysis, lipemia, collection tubes with anticoagulants, specimens beyond their acceptable length of time from collection as listed above, specimen types other than those listed.

Shipping Instructions:

- Label all specimen tubes with two unique patient identifiers (specimen labels are available for your use on NTD's Prenatal Screening Requisition Form).
- Place tube into provided shipping canister and close tightly. Place closed shipping canister into the biohazard bag. Place the biohazard bag into the collection kit box along with the completed test requisition form.
- Ship specimens within 24 hours. Refrigerate specimens that will be delayed more than 24 hours before shipping. Friday shipments must be labeled for Saturday delivery. All specimens must be labeled with patient's name as it appears on the requisition AND either the preprinted requisition number, date of birth or MRN number. A test requisition form must accompany each specimen.
- Ship specimens FedEx **Eurofins NTD, LLC.**
Priority Overnight® to: **80 Ruland Road, Suite 1**
Melville, NY 11747
- For more information about NTD Eurofin's prenatal screening options, or to order test supplies, please contact your Genetics Account Executive, or call us at 1-888-NTD-LABS (683-5227).